# INSTRUCTIONS FOR USE

## Defibrillation Pads with CPR Device

Use Cardiac Science® defibrillation pads with the Powerheart® G5 AED for emergency treatment of sudden cardiac arrest. The AED automatically identifies electrode pads connected to it, and adjusts the maximum shock energy.

- For patients over 8 years of age or heavier than 55lbs (25kg)
- Intended for short-term use (<8hrs)
- Use before the expiration date listed on the package.
- The AED automatically reads the expiration date of the electrode pads, and notifies the user of expired pads.

**WARNING!** Do not reuse pads.

Reuse of pads may result in skin burns, improper AED performance or patient-to-patient contamination.

**DO NOT** open until ready to use.

**DO NOT** use damaged pads.

Failure to remove blue plastic completely could impact therapy delivery. Before use, check cable and connectors for damage.

**DO NOT** pull on the lead wires to separate the pads from the blue plastic. Not intended for use in pacing.

When placing pads:

Place the pad at least an inch from any implanted device.

Ensure patient's skin is clean and dry before use.

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### English

Patents pending.

For a complete list of U.S. and foreign patents, see [www.cardiacscience.com](http://www.cardiacscience.com).

32°F  
0°C

0086

Not made with  
natural rubber latex

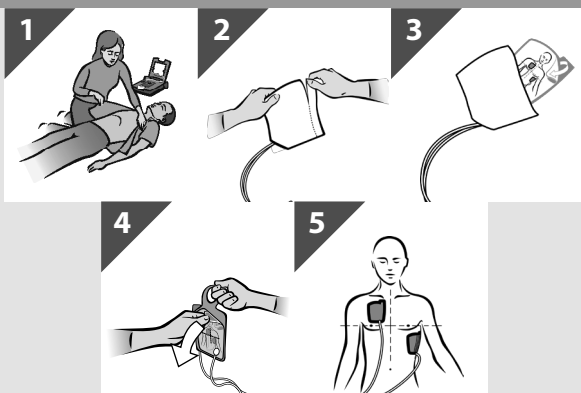
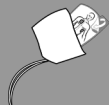
Cardiac Science, Powerheart, the Shielded Heart logo and Rescue Ready are trademarks of Cardiac Science Corporation.

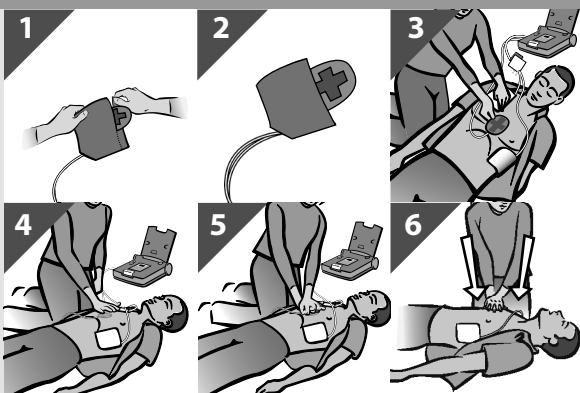
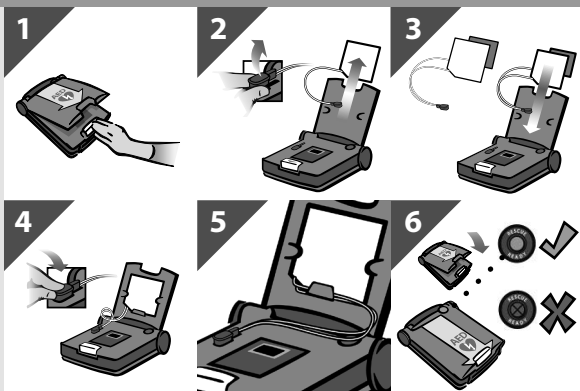
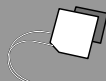
Copyright © 2015 Cardiac Science Corporation.

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All Rights Reserved.
MDSS GmbH